




# ResMed AirSense 10 Air Intake Filter Replacement

Replacing the Air Intake filter on the ResMed AirSense 10 ventilator for normal care.

Written By: Jacey Cobb

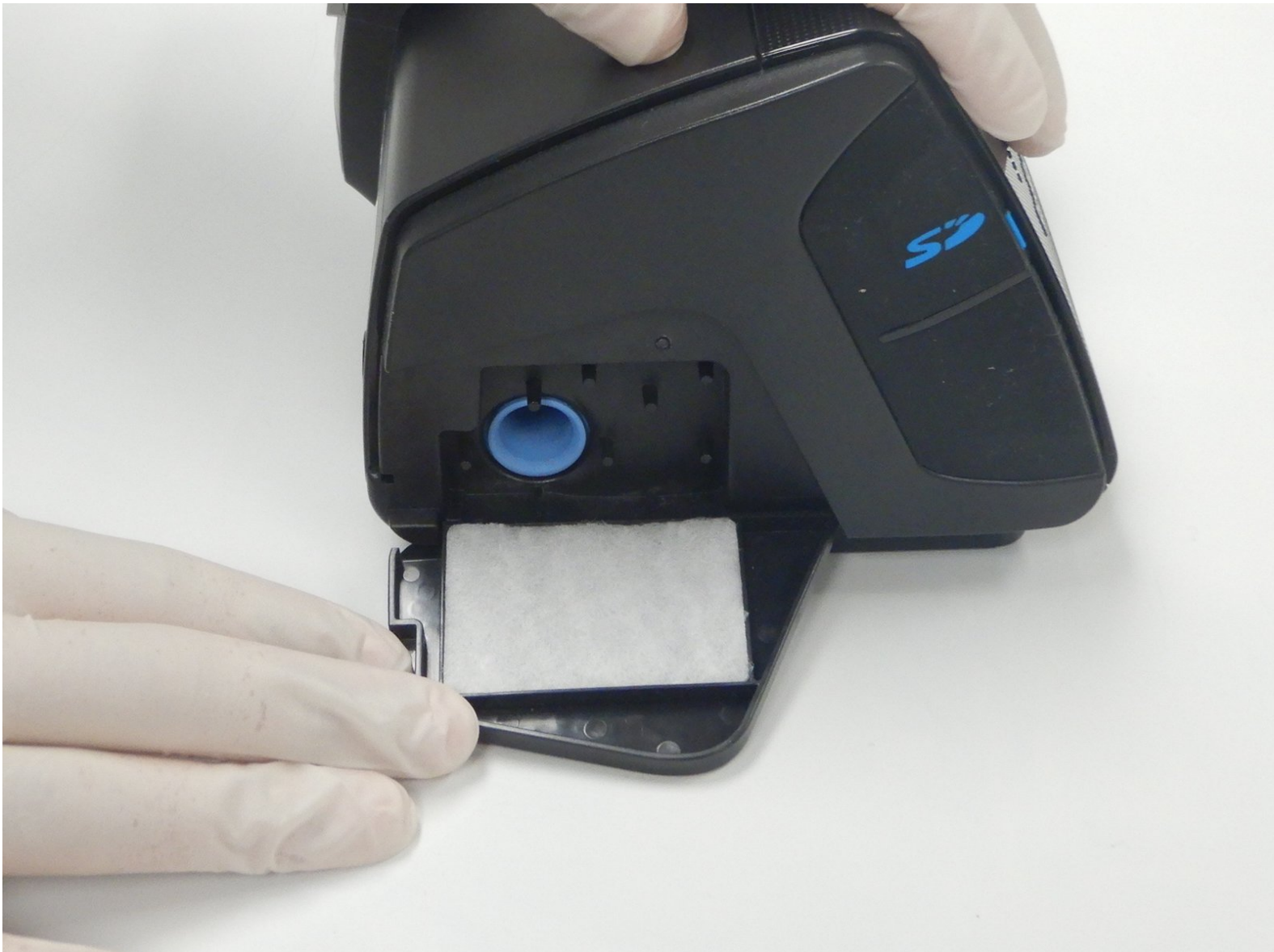

## INTRODUCTION

This guide will explain how to replace the air intake filter in the ResMed AirSense 10 ventilator. Regularly check the intake air filter to prevent the entry of dust and water. Replace the intake air filter every 6 months or if there are any damages or blockages to the filter.



### PARTS:


- [2 1/8" x 1 7/16" Disposable Air Filter](#) (1)
-

## Step 1 — Air Intake Filter



- ⓘ The air filter cover is plastic mesh plate, on the same side as SD card holder.
- Pry open the intake air filter cover on the bottom of the left side panel of the device.

## Step 2

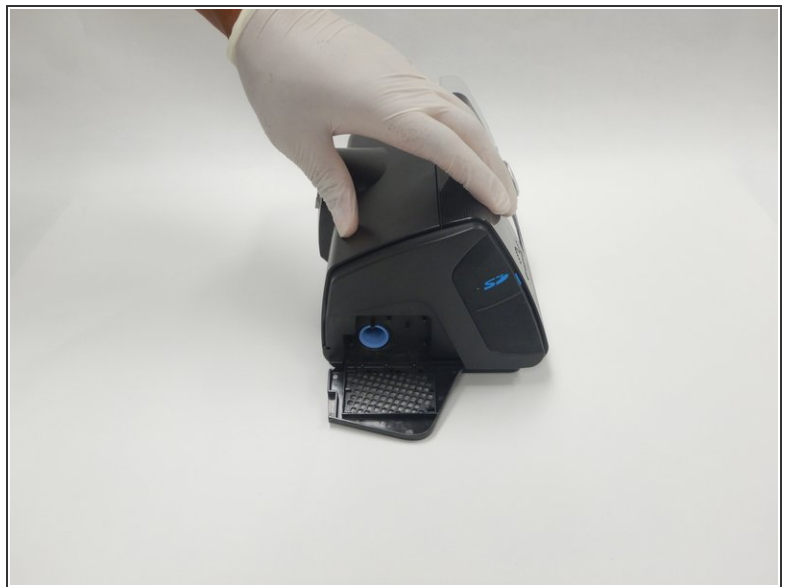

- Remove the used air filter and inspect for damages or blockages.
- Replace the air filter as needed.

To reassemble your device, follow these instructions in reverse order.