




ResMed AirSense 10 Air Outlet Adapter Replacement

This repair guide will help you replace the air outlet adapter on the ResMed AirSense 10 ventilator once the rubber seal is worn.

Written By: Jacey Cobb

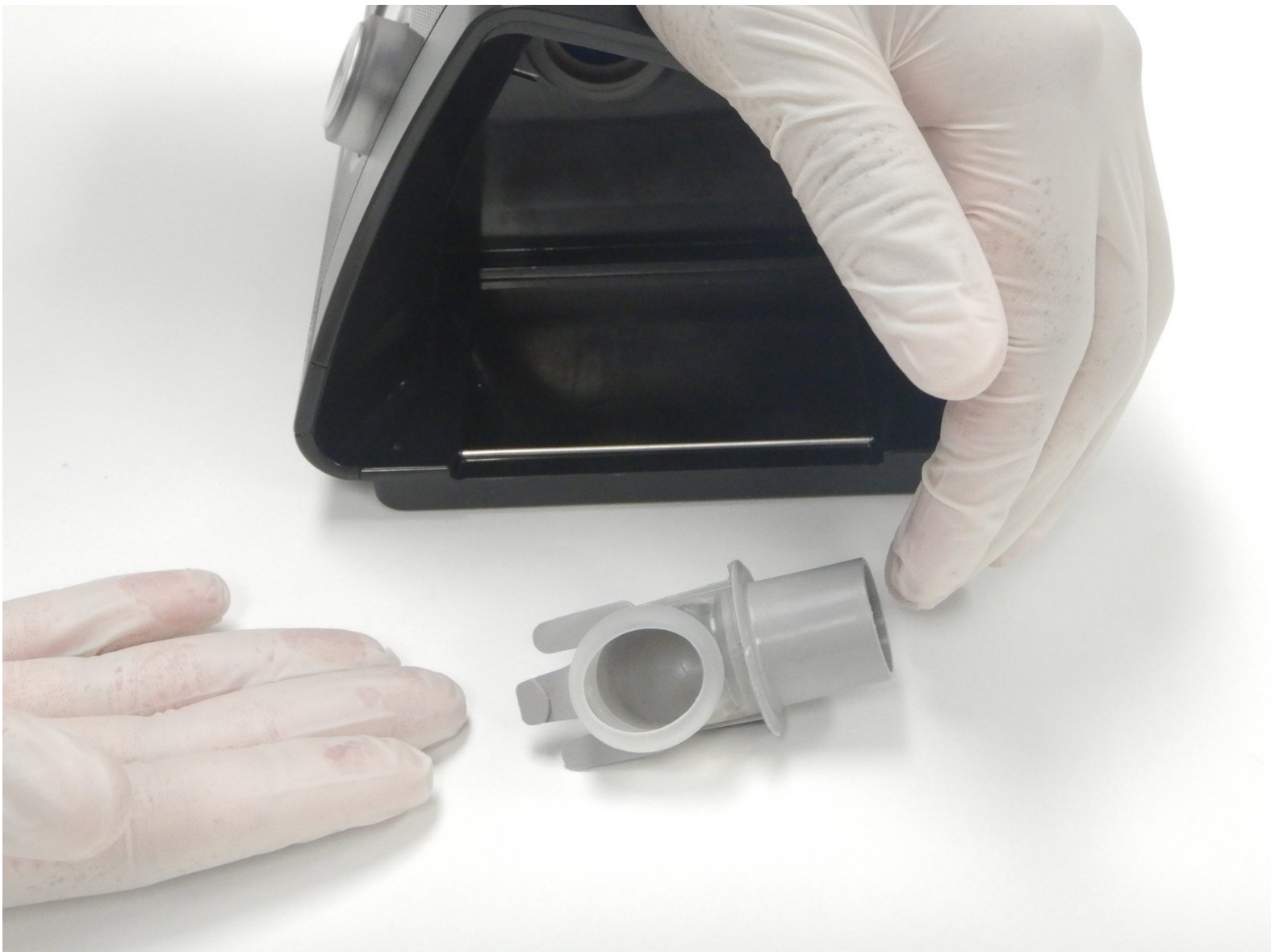

INTRODUCTION

Over time the rubber seal on the air outlet elbow of the ResMed AirSense 10 medical ventilator will wear and cause small air leaks. If the hose and mask are verified to be not leaking, the leak can be located on the air outlet elbow. Check the air outlet elbow when trouble shooting leakages and replace as needed.



TOOLS:


- [Spudger](#) (1)



PARTS:


- [ResMed Airsense 10 Air Outlet adapter](#) (1)

Step 1 — Air Outlet Adapter

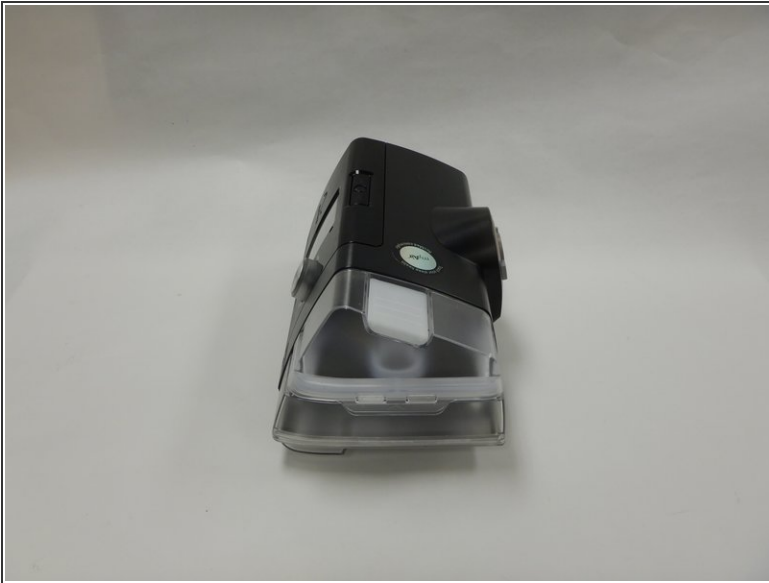

- Position the ResMed Airsense 10 with the water reservoir facing towards you.
- Remove the ResMed water tank.

Step 2

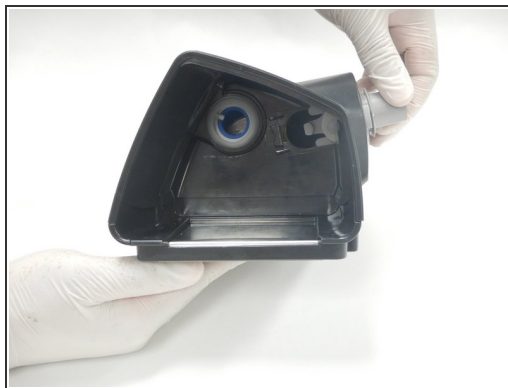

- Use the spudger to disable the lock on the air outlet elbow.
 - Remove the air outlet elbow through the back opening on the ResMed.
- i** The Air outlet elbow is now free and ready to be replaced.

To reassemble your device, follow these instructions in reverse order.

This document was generated on 2022-09-29 11:07:16 AM (MST).

