



ResMed AirSense 10 Top Cover Replacement

Written By: Mitchell Novak



INTRODUCTION

This guide shows how to remove the top cover, and acts as a prerequisite for the Motherboard and the Wireless Communication Board guides.



TOOLS:

- [TR10 Torx Security Screwdriver](#) (1)
 - [iFixit Opening Tool](#) (1)
-

Step 1 — Top Cover



- Remove the water tub by gently pulling it out.

Step 2



- Pry off the face plate by using your finger or an opening tool.
 - ⓘ Use a moderate amount of force when prying open.
- Remove the face plate from the top cover.
 - ⓘ There will be a clicking sound on either side of the divot when assembling and disassembling.

Step 3



- Unscrew and remove the two 11.8 mm TR10 Torx screws that were revealed from the face plate being removed.

Step 4



- Unscrew and remove the two 11.8 mm TR10 Torx screws from the bottom of the device.

Step 5



- Remove the top cover from the rest of the device.

To reassemble your device, follow these instructions in reverse order.